


NOTICE

This shipment of CS labels are **TEMPORARILY** wound **INSIDE** instead of our normal **OUTSIDE** wind. **PLEASE NOTE:** this will be corrected in the very near future.

Below you will find step-by-step instructions that will help walk you through how to load your CS-2 Labeling Device with the temporary labels that are wound inside

As stated earlier, this is a TEMPORARY issue and will be corrected as we deplete our current inventory. We sincerely apologize for any inconvenience this may cause. If any technical issues arise, please contact your sales representative at 1-800-323-4840.

Fig. A

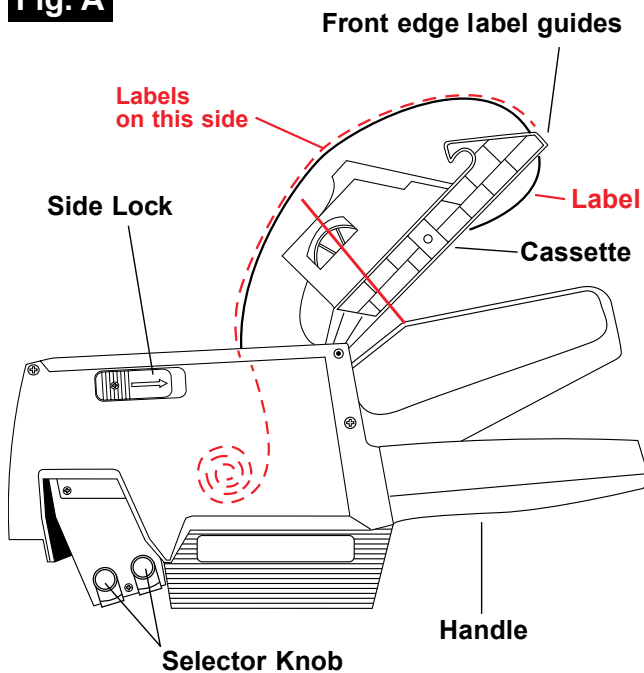


1. To load label gun (Inside Wind)

Remove 7 to 10 labels from leading edge. Press both side locks and open cassette. Insert labels as shown in **Fig. A**. Thread labels under front edge label guides (**Fig. B**) and close cassette until it snaps.

Feed Label between feed drive wheel and roller. Squeeze handle 6 or 7 times. The labels will feed through the gun. Select desired bands. The gun is now ready to use.

Fig. B

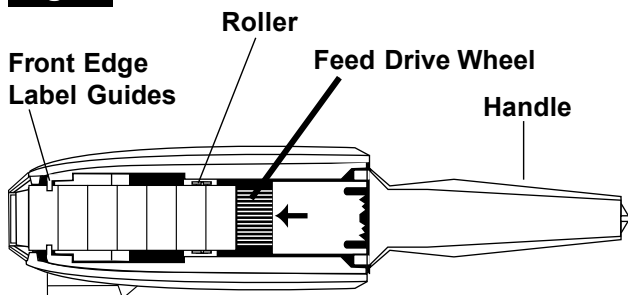

TimeMed Labeling Systems, Inc.

144 Tower Drive

Burr Ridge, IL 60527

Phone: 800-323-4840

Fax: 800-548-5359

www.TimeMed.com